ULTRASONİK HAVA NEMLENDİRİCİ

ULTRASONIC HUMIDIFIER

HNK-HUM-01

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Hünkar Ecza ve Medikal San. Tic. Ltd. Şti

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Index

1.	Warnings and Safety Measures	16
2.	Device Features	19
3.	Device Specifications	19
4.	Device Design and Features	20
5.	Device Usage Instruction	21
6.	Cleaning and Maintenance	23
7.	Storage and Transport	24
8.	Troubleshooting	25
9.	Warranty Conditions and After Sales Support	25
10.	CE Declaration of Conformity	26
11.	Symbols and Definitions	27
12.	Device & Outer Box Symbols and Descriptions	28
13.	Manufacturer and Technical Service Information	28

Thank you for choosing Nimomed® Ultrasonic Air Humidifier. For your own safety and the safety of others, read the user manual carefully before using the device and keep it as a reference for future use.

1. Warnings and Safety Measures

a) Warnings About Instructions for Use

WARNING!

This sign identifies hazards that can result with injuries or death

ATTENTION!

This sign identifies hazards that can cause malfunction of the product or become damaged.

b) Usage Area

This appliance is intended for domestic use. Otherwise, malfunctions may occur in the device.

c) Security Warnings

- ! Read the user manual before using the device.
- ! This device is not suitable for people who are mentally unstable or for people who do not have enough knowledge to use such devices. Do not allow this type of person to use the device.
- ! The device is not a toy. Therefore, do not allow children to use the device and be more careful when using the device with children.
- ! Keep the items such as plastic bag and cardboard away from children when you take them out of the package. Otherwise, children may harm themselves by swallowing these substances, especially playing with packing bags may pose a choking-swallowing risk.
- ! Always unplug the device before installing, removing, cleaning or filling it with water.
- ! Before operating the device, make sure that the mains voltage is the same as shown on the type label of your device.

- ! Do not use your device in or near bathtubs, sinks, etc. that are absolutely filled with water. Never immerse the device in water.
- ! Prevent the cord, plug or the main body of the device from getting wet.
- ! Do not try to open or repair any part of the device. If these instructions are ignored, the user is responsible for any malfunctions that may occur.
- ! If the power cord of the device is damaged, never plug in the device. Do not use the device. Contact immediately the authorized technical service.
- ! All repairs of the device must be carried out by the authorized service.
- ! Do not pull on the cable when unplugging the device.
- ! To unplug the device, always pull the plug of the device.
- ! Do not pull the device cord firmly and do not bend it excessively.
- ! Do not use harsh, abrasive cleaning materials to clean your device.
- ! You can clean the outer surface of your device with a cloth.
- ! Do not clean the evaporator tank in the base unit of your device with detergent.
- ! In case of unusual sound or odor, unplug the product as soon as possible.
- ! Do not touch the liquid in the water or evaporator tank during operation. Risk of electric shock.
- ! If there is no liquid in the water tank, never operate the device. Always make sure that there is water in the reservoir before operating the device.
- ! While the humidifier is operating, the water tank must be placed in the base unit.
- ! Use clean water to fill the tank. In case the water is lime, distilling or softening the water will reduce the limescale inside the device. Thus, the device will outlast.
- ! Always operate the device on a firm hard surface.
- ! Never plug the device into the socket before the parts are fully assembled.
- ! Do not block the air vents or insert any objects into the vents.
- ! Do not move, shake, bend the device or do not try to empty the water while device is operating.
- ! You can open the lid on the water tank and add water with the help of a cup while the device is operating.
- ! Since the device generates humidity, it may cause condensation on the walls and furniture by air. The device should not be left in unattended in closed rooms while working.
- ! Always leave the door of the room open while the device is operating.

- ! Keep the water tank cap closed while the device is operating. Avoid blowing vapor directly on to children, walls or objects by changing the direction of the vapor outlet.
- ! Do not cover the device while it is running, do not cover the vapor outlet.
- ! Do not put aromatherapy oil or any substance into the vapor outlet, water or evaporator tank.
- ! Do not try to disassemble or open any part of the humidifier.
- ! Turn off your device and unplug it after each use. Do not plug and unplug the device with wet hands.
- ! Keep the device away from the children, do not allow to play with the device.
- ! This device can only be used by children over 8 years of age. People who are physically, sensually limited or lacking knowledge should not use the device unsupervised.
- ! Turn off the device and unplug the device during cleaning.

2. Device Features


360-degree adjustable vapor outlet
Variable vapor output speed
Automatic shutdown when water runs out
Removable water tank
Silent operation

3. Device Specifications

Product Name	Ultrasonic Air Humidifier
Model Number	HNK-HUM-01
Operating Method	Ultrasonic Vibrating Piezo
Nominal Operating Voltage	220 – 240 VAC 50/60Hz
Power Consumption	30 W
Vapor Speed	250 ml/hour
Water Tank Capacity	4.5 liter
Working Environment Conditions	Temperature: 5°C-40°C Humidity: <80%R.H. Atmospheric Pressure:86-106 kPa
Storage and Transport Conditions	Temperature: -20°C-55°C Humidity: <80%R.H. Atmospheric Pressure:70-106 kPa
Electrical Safety Class	Class II
Solid and Liquid Protection Class	IP21
Noise	Less than 45 dB
Sizes	20cm(W)x20cm(L)x34cm(H)
Weight	918 gr

4. Device Design and Features

a) Device Image and Parts

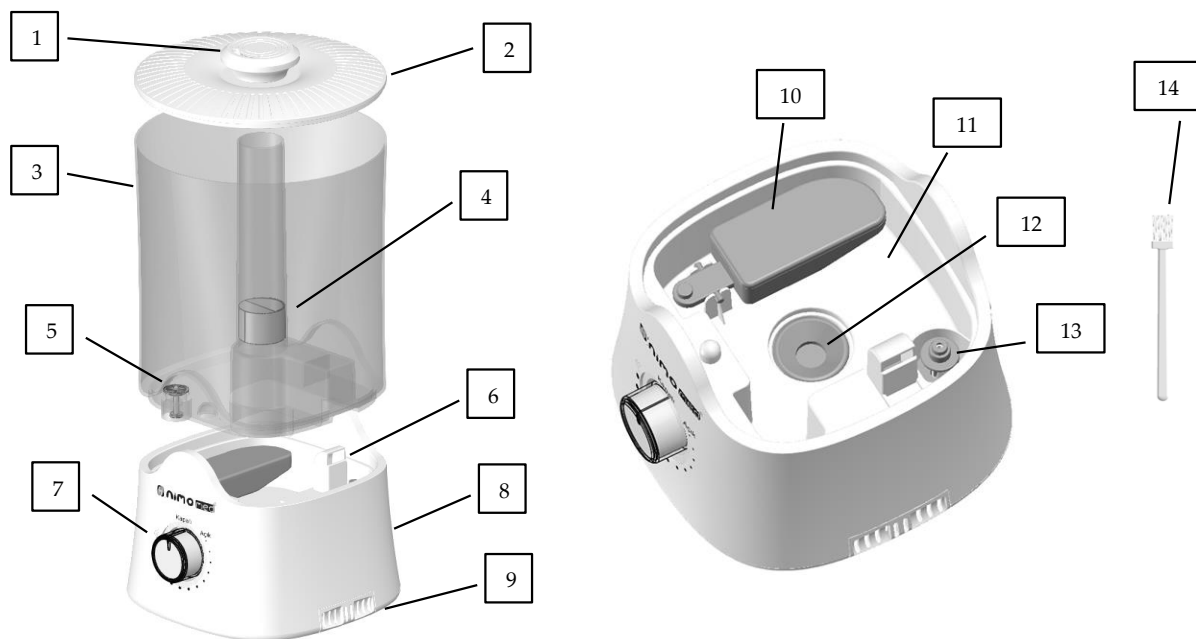Part Number	Part Name	Part Number	Part Name
1	Adjustable Directional Vapor Outlet	8	Base Unit
2	Water Tank Cover	9	Air Filter
3	Water Tank	10	Water Flow Float
4	Drop Separator	11	Evaporator Tank
5	Silicone Valve	12	Ultrasonic Evaporator
6	Air Blow Hole	13	Safety Float
7	On-Off, Vapor Speed Control Button, LED Indicator	14	Cleaning Brush

b) Light Indicator and Meanings

No Light

Device Off

Blue Light

Device is Operating

Red Light

There is not Enough Water in
the Water Tank (Fill water)


5. Device Usage Instruction


Warning!

Do not open the device in case of a possible malfunction.

- 1) Place the device on a flat hard floor.
- 2) Remove the devices water tank cap.
- 3) As shown in the figure below, fill the water reservoir of the device with the help of a container to a level that does not overflow. If desired, the water tank of the device can be removed from the evaporator pool and filled directly under the tap.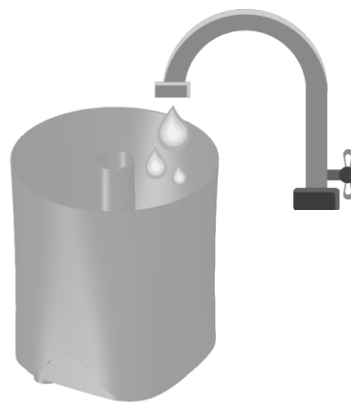

- 4) Place the water tank cover of the device on the water tank.
- 5) Plug the device into the socket.
- 6) Turn the vapor speed control knob clockwise to turn it on.
- 7) Vapor output will be observed from the adjustable vapor outlet after a few seconds.
- 8) Vapor speed can be increased according to the desired level by turning the vapor speed control knob clockwise.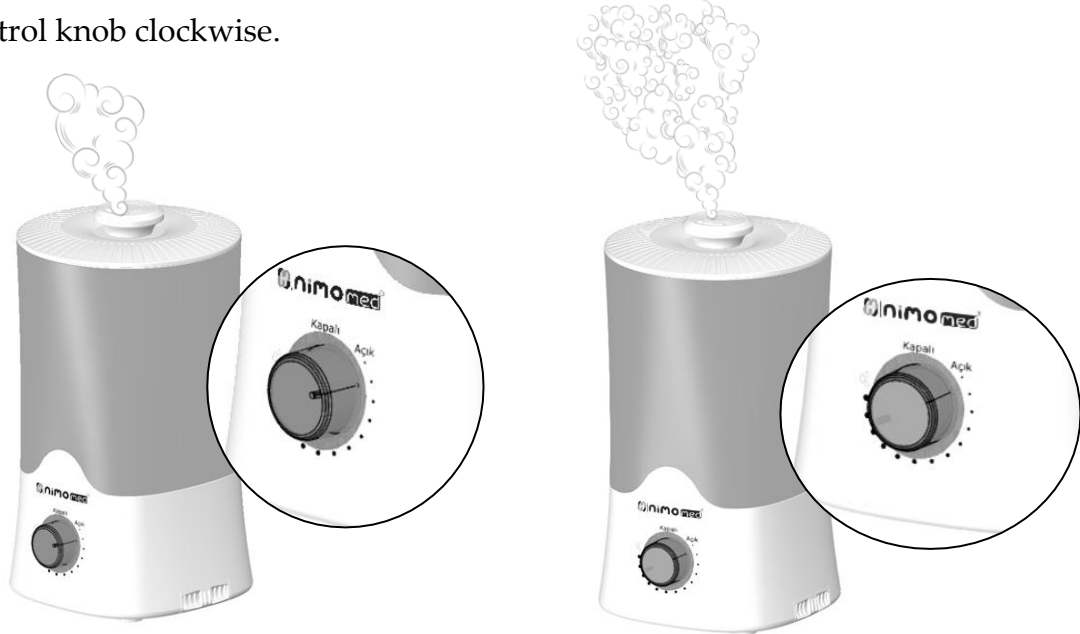


- 9) After using the device, turn it off and unplug it.

NOTE: It is normal to hear a dripping noise when operating high level vapor.

NOTE: Water can be supplemented while the device is operating. Open the top cover and add water to the water tank. Water can be added by replacing the cover without removing the water tank.

⚠ WARNING!

Add water into the water tank of the device, do not add it directly to the vapor outlet or evaporator tank.

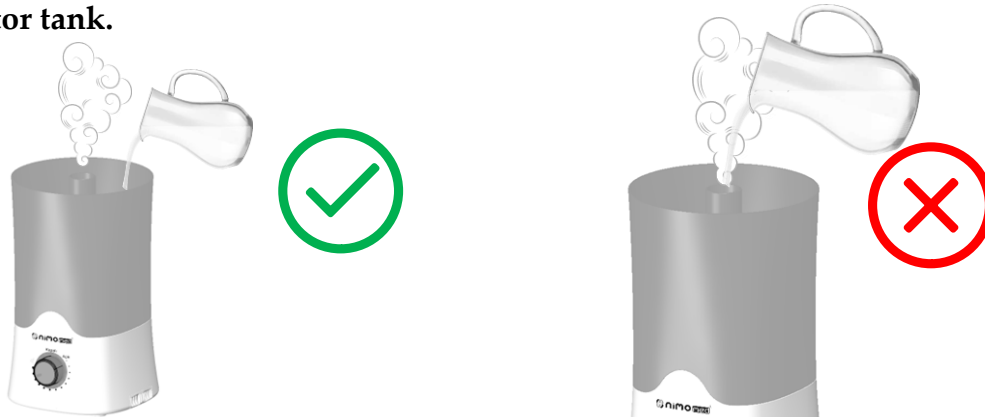6. Cleaning and Maintenance

- 1) Unplug the device before cleaning.
- 2) Empty the remaining water in the base unit of the device.
- 3) Wipe the base of the device with a damp cloth.
- 4) In order to avoid scratches, do not use abrasive materials while cleaning the device.

Cleaning the Water Tank

- 1) Fill the water tank with water.
- 2) Put a little washing-up liquid in it and shake it well.
- 3) While shaking the water tank, hold it with both hands, do not tighten excessively to prevent cracking.
- 4) After shaking the water tank, pour the water inside and rinse the tank thoroughly.

Cleaning of Evaporator Tank and Ultrasonic Evaporator

- 1) To remove water and lime residues, pour water into the evaporator pond so that it does not overflow.
- 2) Add 2 tablespoons of white vinegar to the water.
- 3) Leave the prepared mixture for at least half an hour.
- 4) Apply the solution to the areas with ultrasonic evaporator and lime residue with the help of the brush or soft bristle brush which comes with the device.
- 5) Pour the solution and rinse the evaporator tank of the device.

NOTE: It is recommended to clean the water tank once a week. It will prevent the formation of bacteria and odors.

NOTE: Ultrasonic evaporator and evaporator tank should be cleaned once a month to prevent residue accumulation.

Cleaning of Air Filter

- 1) Unplug the device before cleaning the filter.
- 2) Drain the remaining water in the base unit of the device.
- 3) Pull out the air filter of the device and remove it.
- 4) Check the air filter periodically, if dust or dirt appears, clean and replace the filter.

NOTE: Optionally, you can add aromatic oils or fragrances into the air filter, allowing the device to give pleasant scents to your room during operation.

7. Storage and Transport

- If you are not going to use the humidifier for 2 days or more, empty the water in the water and evaporator tank.

NOTE: It is normal for the device to make a clicking noise from the water float valve while draining the water in the evaporation chamber. Counterweight inside the device's float can make sounds during shaking.

- Dry the remaining water with a dry cloth.
- Leave the water and evaporator tank open to let the device completely dry.
- To prevent algae and odors, make sure that there is no water inside the device.

During Transport and Shipping

- Do not drop the product.
- Avoid impacts.
- Take precautions to ensure that it is not jammed and crushed.
- Follow the markings on the packaging.
- Carry the device in his original packaging.

Box Content

- Nimomed HNK-HUM-01 Ultrasonic Air Humidifier (1 Piece)
- Cleaning Brush (1 Piece)
- User Manual (1 Piece)
- Warranty Certificate (1 Piece)

8. Troubleshooting

Problem	Solutions
Air Humidifier Does Not Work	☞ Make sure the device is properly plugged in. ☞ Make sure the vapor control knob is on.
The light indicator lights up, but there is no vapor output or very little vapor is emitted.	☞ Make sure the adjustable vapor outlet is not blocked. ☞ Make sure there is enough water in the water tank.
There is enough water in the water tank, the device gives a red warning and there is no steam output.	☞ Check that there is enough water in the evaporator pool by pulling the water tank upwards. ☞ Remove the lime, sediment, etc. in the water by moving the water flow float up and down with your hand. Make sure it's not stuck with anything. ☞ Put the water tank back on. Make sure that water from the water pool fills the evaporator chamber.
A white dust forms on and around the device.	☞ There may be too much mineral and lime in the water you use. Try to use purified or filtered water.
There is a bad smell while the device is operating.	☞ Clean the water and evaporation tank.

ATTENTION!

If the suggested solutions do not solve the problem, never try to repair the device by yourself and contact the nearest authorized service.

9. Warranty Conditions and After Sales Support

The warranty period of the device is 2 years from the date of purchase.

- ! If the device has been opened, damaged or attempted to be repaired by the user, it is not covered by the warranty.
- ! NOTE: Please do not forget to take and keep your sales receipt with the warranty certificate.

10. CE Declaration of Conformity

This device fully complies with the European Union's 2014/30 / EU Electromagnetic Compatibility Directive and 2014/35 / EU Low Voltage Directives.

We declare that this device complies with the basic safety and essential performance set by the following standards.

2014/35/EU LVD Standards

EN 60335-1:2012 + A11:2014

EN 60335-2-98:2005 + A1:2006 + A2:2010 + A11:2019

2014/30/EU EMC Standards

EN 55014-1:2017

EN 55014-2:2015


EN 61000-3-2:2014

EN 61000-3-3:2013

11. Symbols and Definitions

	CE Marking
	Manufacturer Company Information
	Production Date
	Class II Device
	Read the User Manual
	LOT Number
IP21	Device with IP code: Protects the device against solid objects larger than 12.5 mm and vertical dripping liquid.
	Product Model Number
	According to the 2002/96 / EC electronic and electrical appliances directive, it is forbidden to dispose the device and it's parts after the service life is completed. Local authorities should be consulted for its disposal.

12. Device & Outer Box Symbols and Descriptions

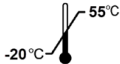	Keep Away From Sunlight
	Protect the Device from Getting Wet
	Fragile Product Carry Carefully
	Operating Temperature Range: 5 ° C - 40 ° C
	Storage and Transport Temperature Range: -20 ° C - 55 ° C
	Operation, Storage and Transportation Environment Humidity Range: <80% R. H.
	Device Barcode Number

13. Manufacturer and Technical Service Information

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