21\*11.5CM

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**For maximum pleasure, follow these instructions:**

- 1.- Open the packet and remove the condom carefully as you could damage it with your nails, teeth or pointed objects (jewellery, piercings, etc.)
- 2.- Apply the condom preferably with the penis erect before vaginal penetration to avoid pregnancy or becoming infected with sexually transmitted diseases.
- 3.- Make sure that the rim of the condom is on the outside as the condom is unrolled over the penis, and, at the same time, hold the end of the condom so that no air remains trapped inside.
- 4.- Unroll the condom fully downwards before sexual activity and, in the event that it comes off during sexual activity, use another.
- 5.- After ejaculation, carefully hold the condom at the base of the penis and remove it by pulling from the end making sure not to spill its contents or touch your partner's genitals or mucosa with fingers that have come into contact with sperm.

**ADVICE AND PRECAUTIONS**

- 1.- Never use a condom more than once. Use a new condom for each sexual intercourse.
- 2.- If you change partner during a sexual act, always use a new condom, otherwise you may transmit a sexually transmitted disease from the first to the second partner.
- 3.- If the material that the condom is made of looks broken or is visibly damaged, do not use it as there is a serious danger of splitting or leakage.
- 4.- In the event of splitting, the semen may come into contact with your partner's vagina, and so, if the condom has been used in order to prevent pregnancy, you are advised to seek appropriate medical or pharmaceutical attention within 72 hours from a family planning clinic, hospital or chemist/pharmacy and ask for the morning after pill.
- 5.- If you require additional lubrication, only use water-based lubricants, such as glycerine or lubricating jellies, but do not use oil-based lubricants, such as vaseline, baby oils, massage oils or body lotions, butter, margarine, etc., as they may damage the condom.
- 6.- Consult your doctor or chemist/pharmacist to find out about the compatibility of latex condoms with topical products applied to the penis or the vagina.
- 7.- Warning: do not use this product if you are allergic to latex.
- 8.- When used properly, condoms help prevent many sexually transmitted diseases such as, syphilis, gonorrhoea, chlamydia, genital herpes, hepatitis B and AIDS, but they do not completely remove the risks involved.
- 9.- Non-vaginal use of the condom, contact with teeth, shaved hair or piercings all increase the risk of the condom splitting. Therefore, take extra precautions in those cases.

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