

KESİNLİKLE KAYNAR SU KULLANMAYINIZ! Never use boiling water!

BUYOT KULLANMA TALİMATI

Buyotu kaynar suyla değil, sıcak su ile maksimum (2/3) kapasiteye kadar doldurunuz. Suyu soğuk su musluğundan alıp ısıtınız.

Kaynar su, buyotunuza zarar vereceği için kullanmayınız. Daha yüksek oranda bakır içerdiğinden direk sıcak su musluğundan alınan su buyotunuzun ömrünü kısaltır; kullanmayınız.

Buyotunuzu dikey olarak ve boyun kısmından tutarak doldurunuz.

Açılan boyun kısmında su belirinceye kadar buyotun düz bir zeminde eğerek havayı dışarı çıkartınız. Daha sonra sızıntı olmadığına emin olmak için tıpayı sıkıca kapatınız. Elinizle sıkmanız yeterlidir.

Buyotun sıcak yüzeylerle temasını engelleyin. Buyotunuza zarar verebileceğinden yağ ile temasından kaçınınız.

Kullanmadığınızda tamamen süzün serin ve kapalı mekanda tıpasını çıkararak saklayınız. Direk güneş ışığından koruyunuz. Buyotun üzerine herhangi bir şey koymayınız. Tıpayı kullanmadan önce kontrol ediniz. Kullanım talimatlarına uyunuz.

INSTRUCTIONS FOR USE

When filling this hot water bottle do not use boiling water and fill to a maximum of two-thirds capacity. Hold the bottle by the neck in a upright position and fill slowly to avoid hot water splashing back.

Expel air from the bottle by lowering it carefully onto a flat surface until water appears at the opening. Screw the stopper sufficiently tight to ensure that there is no leakage. Finger tight should be adequate.

Do not fill using water from the domestic hot water System as this can considerably shorten the life of the hot water bottle.

Prevent the bottle from contact with the hot surfaces. Prevent contact with oil and grease.

When not in use drain completely and keep with the stopper removed, in a cool dry dark place. Prevent exposure to sunlight. Do not place anything on top of the bottle. Check the bottle stopper for wear and damage at regular intervals. Retain these instructions for future reference.



Hünkar Ecza Ve Medikal San. ve Tic. Ltd. Şti
Akçeşme Mah. 2019 Sk. No:5
Merkezefendi DENİZLİ / TURKEY
+90 258 371 46 56
www.hunkarmedikal.com